

# FLOCARE BOLUS SET

## INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

You will need:


- Flocare Bolus Set
- Feed as prescribed
- Water for flushing
- A 60ml syringe

Check all expiry dates and wash hands thoroughly


Open the packaging and lay contents on a clean dry surface



Attach a 60ml syringe to the bolus set

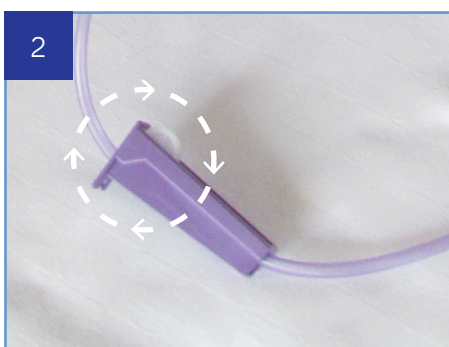


Open the roller clamp and flush with water as directed to expel any air

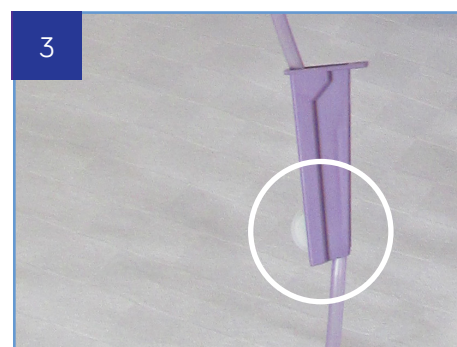


Close the roller clamp




Remove the syringe from the bolus set

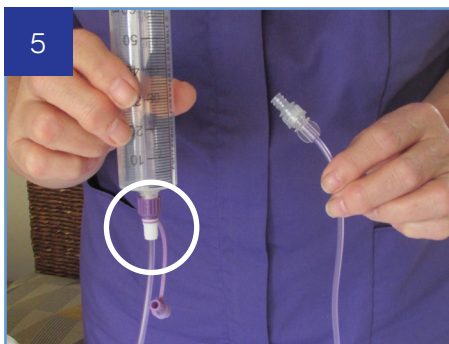

Attach the barrel of the bolus set to the extension set

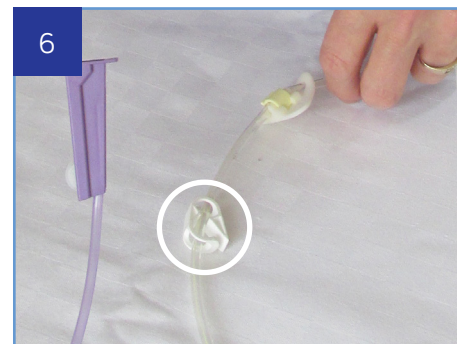


Close the clamp on the feeding tube (if appropriate)

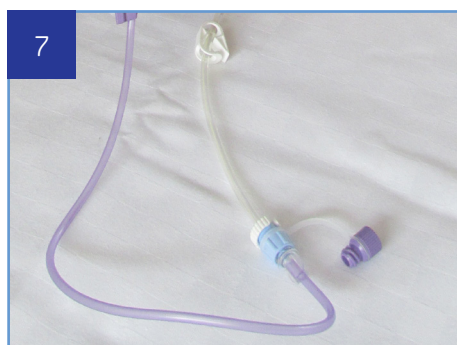

Attach the extension set to the feeding tube



Pour the prescribed feed into the barrel and open the roller clamp

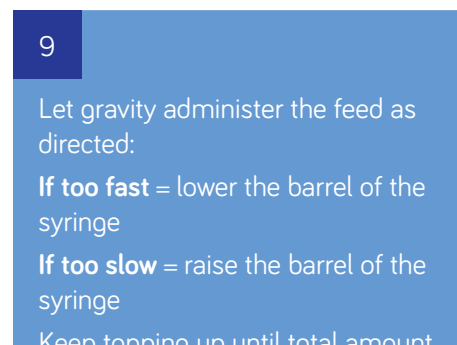

Let gravity administer the feed as directed:


**If too fast** = lower the barrel of the syringe

**If too slow** = raise the barrel of the syringe


Keep topping up until total amount of prescribed feed has been administered

# FLOCARE BOLUS SET

## INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

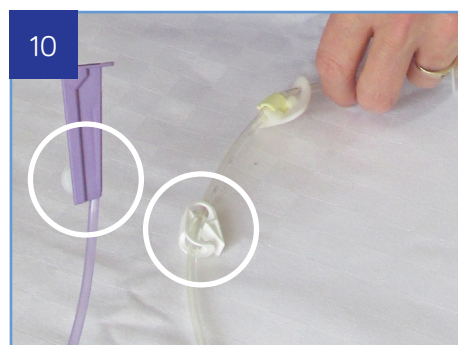

Close the roller clamp and the clamp on the feeding tube (if appropriate)

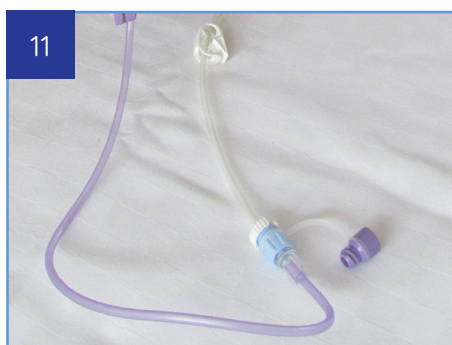

Remove the bolus set from the feeding tube



Attach the syringe, open the clamp on the feeding tube (if appropriate) and flush with water as directed

Discard bolus set as advised by your healthcare professional or Nutricia Homeward Enteral Nurse Specialist

**The following cleaning instructions must be followed immediately after use of a bolus set.**

❶ Do not allow traces of feed to dry in the bolus set as they might not be removed by cleaning

❷ Use warm soapy water to wash the syringe and the tube of the bolus set

❸ Rinse thoroughly under cold running tap water to remove all traces of the soapy water

❹ Allow to air dry

❺ Once completely dry store in a clean dry place

❻ The bolus set may be cleaned and re-used for no longer than 24 hours