

FLOCARE Z-STAND

INSTRUCTIONS FOR USE – OPTRI BOTTLE



1
Attach Flocare Infinity pump to Z-Stand



2
Attach Flocare Infinity Pack giving set to prescribed product



3
Using hanging eye on OpTri bottle (or Z-Hook if available), hang OpTri bottle from rear of Z-Stand. For 1 litre OpTri bottle, hang from the TOP and rear of Z-Stand using a Z-Hook



4
Open door and insert giving set



5
Close door (click)



FLOCARE Z-STAND

INSTRUCTIONS FOR USE – PACK



1 Attach Flocare Infinity pump to Z-Stand



2 Attach Flocare Infinity Pack giving set to prescribed product



3 Using the hole at the top of the Pack hang from rear of Z-Stand



4 Open door and insert giving set



5 Close door (click)

