

# CLEANING INSTRUCTIONS FOR THE BUTTON EXTENSION SET

To reduce the risk of contamination wash the button extension set thoroughly after every use, using the following instructions.

- Close the clamp and turn the extension set anti-clockwise to match up the markings. Gently pull the extension set to remove it from the button
- Open the clamp and wash thoroughly in warm soapy water
- Using a brush, clean the connectors to remove any build up in the threads
- Flush with at least 60ml sterile/cool boiled water to remove any soap remnants
- **If using for more than 7 days (maximum 14 days)** flush with at least 60ml sterilising solution, suitable for use with plastic products. Rinse thoroughly with sterile/cool boiled water to remove any sterilising solution
- Allow to air dry in a hanging position. Store in an air tight container or zip lock bag until next use.

**Before each use check the connectors and tube for cracking, dislocation, or discolouring. If any of these are observed a new extension set must be used.**

