

FLOCARE Z-STAND

INSTRUCTIONS FOR USE – OPTRI BOTTLE



Attach Flocare Infinity pump to Z-Stand



Attach Flocare Infinity Pack giving set to prescribed product



Using hanging eye on OpTri bottle (or Z-Hook if available), hang OpTri bottle from rear of Z-Stand. For 1 litre OpTri bottle, hang from the TOP and rear of Z-Stand using a Z-Hook



Open door and insert giving set



Close door (click)



FLOCARE Z-STAND

INSTRUCTIONS FOR USE – PACK




Attach Flocare Infinity pump to Z-Stand



Attach Flocare Infinity Pack giving set to prescribed product



Using the hole at the top of the Pack hang from rear of Z-Stand

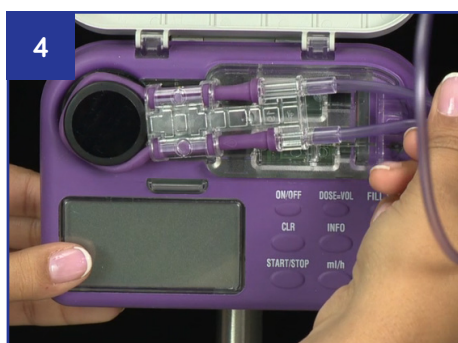

Open door and insert giving set



Close door (click)

