

# FLOCARE BOLUS SET

## INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

You will need:


- Flocare Bolus Set
- Feed as prescribed
- Water for flushing
- A 60ml syringe

Check all expiry dates  
and wash hands  
thoroughly


Open the packaging and  
lay contents on a clean  
dry surface



Attach a 60ml syringe  
to the bolus set

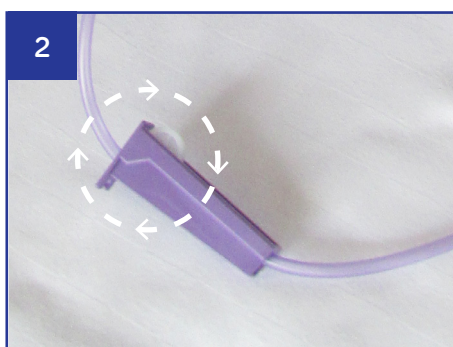


Open the roller clamp and flush with  
water as directed to expel any air

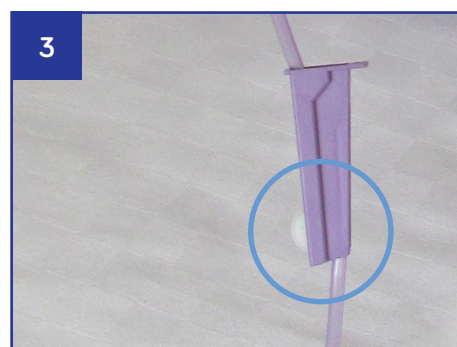


Close the roller clamp




Remove the syringe  
from the bolus set

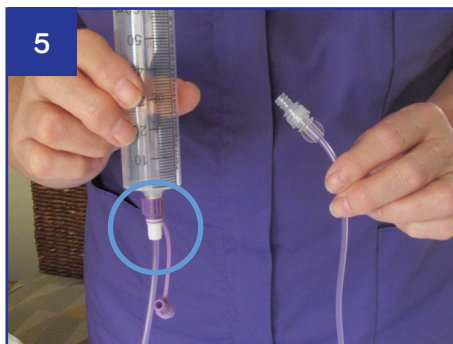

Attach the barrel of the bolus  
set to the extension set

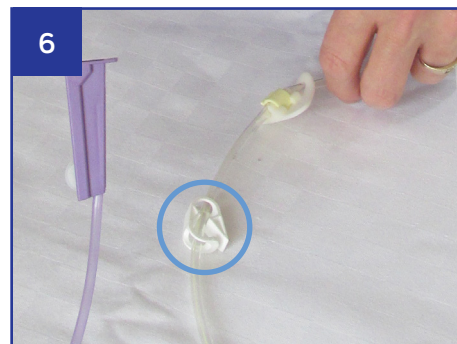


Close the clamp on the feeding  
tube (if appropriate)

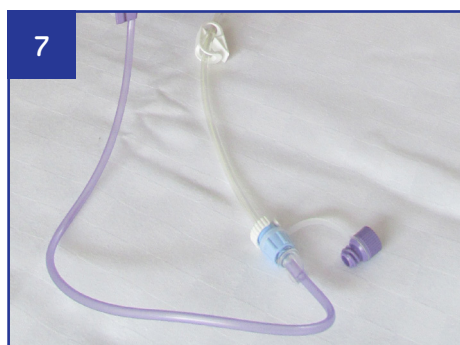

Attach the extension set  
to the feeding tube



Pour the prescribed feed into the  
barrel and open the roller clamp

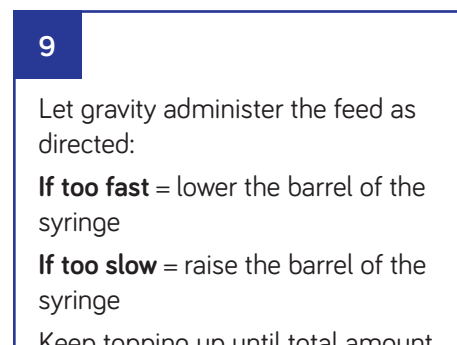

Let gravity administer the feed as  
directed:




**If too fast** = lower the barrel of the  
syringe

**If too slow** = raise the barrel of the  
syringe

Keep topping up until total amount  
of prescribed feed has been  
administered

# FLOCARE BOLUS SET

## INSTRUCTIONS FOR USE & CLEANING INSTRUCTIONS

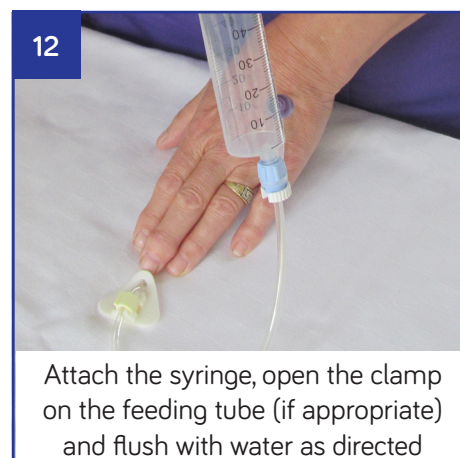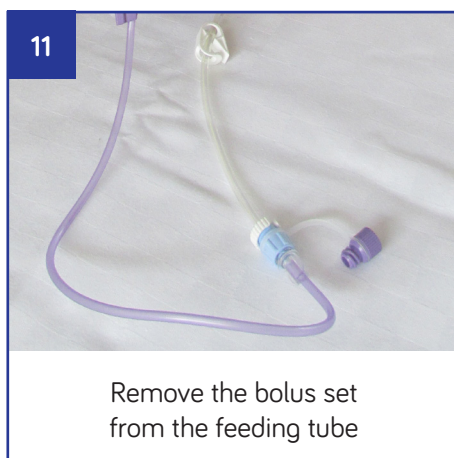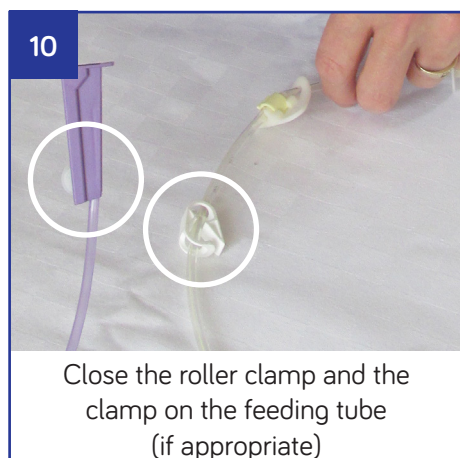

Discard bolus set as advised by your healthcare professional or Nutricia Homeward Nurse Specialist

**The following cleaning instructions must be followed immediately after use of a bolus set.**

❶ Do not allow traces of feed to dry in the bolus set as they might not be removed by cleaning

❷ Use warm soapy water to wash the syringe and the tube of the bolus set

❸ Rinse thoroughly under cold running tap water to remove all traces of the soapy water

❹ Allow to air dry

❺ Once completely dry store in a clean dry place

❻ The bolus set may be cleaned and re-used for no longer than 24 hours