

USING YOUR FLOCARE GIVING SET WITH A FLOCARE CONTAINER

Reducing how much plastic we use has become more important to all of us and we would like you to join us on this mission.

Independent tests on our Flocare giving sets and containers show that using the same giving set for a full 24-hour period is safe to do. Even if you have to mix up more than one feed during the day, you can continue with the same Flocare giving set and container.


This is how you can do it:



All the giving sets and containers are tested and suitable for single patient use. This means that the same person can use the same giving set more than once within 24 hours.



When connecting the giving set for the first time in the day, make sure you keep the cap from the tube connection end of the giving set.

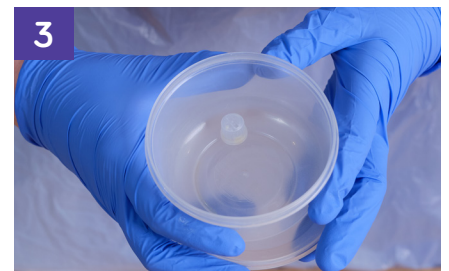

Keep the cap safe in a small container with a lid. The small cap is a choking hazard, so please ensure it is kept safe.



When it's time, disconnect the giving set from your feeding tube.



Replace the cap on the end to protect it from getting dirty.



If the next feed is **not** due within the next 30 minutes, store the **empty** Flocare container and giving set in the fridge.

Continued overleaf...

Continued



When it's time for your next feed, remove the empty Flocare container and giving set from the fridge and decant/mix the new feed as usual without disconnecting the giving set from the container cap.



Every time you connect and disconnect, make sure to replace or remove the small cap and keep it safe.



The empty Flocare container and attached giving set should be stored in the fridge in between each use.



Make sure you discard any left-over nutrition and the giving set after 24 hours.



Remember to put the Flocare container in your plastic recycling bin.

Please speak to your managing healthcare professional before commencing this practice.

If you do not have access to a fridge, you can still reduce the number of Flocare containers and giving sets you use per day. If you would like to know more about this or have any questions, contact your managing healthcare professional.



Nutricia Ltd, White Horse Business Park, Trowbridge, Wiltshire BA14 0XQ.
nutriciaflocare.com

SCC4143-10/22



Scan here to watch a video on how to reduce your plastic usage