

# FLOCARE<sup>®</sup> BOLUS ADAPTOR\*

## INSTRUCTIONS FOR USE

### YOU WILL NEED:


- Flocare Bolus Adaptor
- Tube feed, as prescribed
- Water for flushing
- A 60ml syringe

Check all expiry dates and wash hands thoroughly

Open the packaging and lay contents on a clean dry surface



1

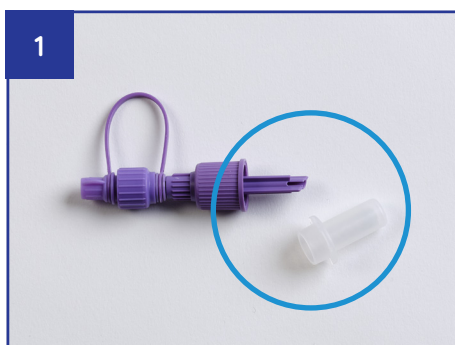

Remove the dust cap from the bolus adaptor

2



Remove the dust cap from the prescribed tube feed

3



Insert bolus adaptor into bottle and screw firmly to ensure there is a tight seal

4



Remove cap from the bolus adaptor. Flush the feeding tube with prescribed amount of water

5



Attach the 60ml syringe to the bolus adaptor

6



Using the plunger of the syringe, gently withdraw the prescribed amount of feed

7

Attach the filled syringe onto the end of the feeding tube or extension set. Push the plunger gently to administer feed and repeat until prescribed amount has been administered

8

Flush the feeding tube with the prescribed amount of water

9



Reseal tube feed bottle by closing the bolus adaptor & store as directed

\*Please note, this item is single use only