



# THE FLOCARE RANGE OF ENTERAL FEEDING PUMPS, TUBES AND ACCESSORIES

This information is intended for healthcare professionals only.



## Enteral Feeding Pumps and Giving Sets

Flocare Infinity™ II Pump

Flocare Infinity™ III Pump

Flocare Giving Sets

## Flocare Feeding Tubes

Nasogastric Tubes (Polyurethane)

Replacement Balloon Gastrostomy Tubes

Gastrostomy Button

Bengmark Naso-Intestinal Tubes

## Flocare Accessories

Flocare Infinity Go Frame

Flocare Infinity Go Bag

Flocare Infinity Z-Stand

Flocare Infinity Bag

Flocare Container

Flocare Universal Adaptor

Bolus Adaptor

Bolus Set

Bolus Set Hanger

Extension Set

Button Extension Set

2-Pack Connector

## Equipment Examples

Flocare Infinity Pack giving set

Bolus feeding

2-Pack connector

## Special Instructions

Button extension set cleaning instructions



All Flocare equipment is compliant with ENFit standard (ISO80369-3).

## SYMBOLS



DEHP-free



CE certificated medical device



Water ingress protection rating



Compatible with the Flocare Infinity enteral feeding pump



For feeding into the stomach



For feeding into the jejunum



Also suitable for paediatric use



Do not re-use



Maximum 24 hours use



For use up to 6 weeks



Can remain in situ for 6-8 weeks



Compatible with product shown, i.e. Pack, OpTri bottle



Recyclable

## CHARRIÈRE SIZE COLOUR CODING



# FLOCARE INFINITY II PUMP

Portable enteral feeding pump, designed for controlling the delivery rate of enteral feeds into the gastrointestinal tract

- Clear LED display
- Water resistant
- Air bubble detection
- Upstream and downstream occlusion detection
- Flow rate 1-400ml/h in 1ml increments
- Dose feature 1-4000ml in 1ml increments
- Lithium ion battery 3.6V DC 2000 mAh, 6 hour charging time
- 24-hour battery life
- Extensive audio and visual alarms
- 1-touch manual priming to prepare the set
- Keypad can be locked
- Continuous memory of feeding program and volume delivered
- Accuracy flow rate of +/- 5% with appropriate Flocare Infinity pump set
- 0ml/hr default setting
- Size: 144 x 105 x 50 mm

PRODUCT CODE: 40407





# FLOCARE INFINITY III PUMP

Portable enteral feeding pump, designed for controlling the delivery rate of enteral feeds into the gastrointestinal tract


- Colour display
- Water resistant
- Ultrasonic air bubble detection
- Upstream and downstream occlusion detection
- Flow rate 1-600ml/h in 1ml increments
- Dose feature 1-3500ml in 1ml increments
- Lithium ion battery 3.7V DC 3200 mAh, 6 hour charging time
- 24-hour battery life
- Extensive audio and visual alarms
- 1-touch manual priming to prepare the set
- Keypad can be locked
- Continuous memory of feeding program and volume delivered
- Accuracy flow rate of +/- 5% with appropriate Flocare Infinity pump set
- Unique information button with onscreen advice
- Size: 149 x 119 x 49 mm

PRODUCT CODE: 121825




# NUTRICIAFLOCARE.COM

The Flocare website contains clear and easy to follow educational resources, expert advice on using the pumps safely, and an interactive online training tool.

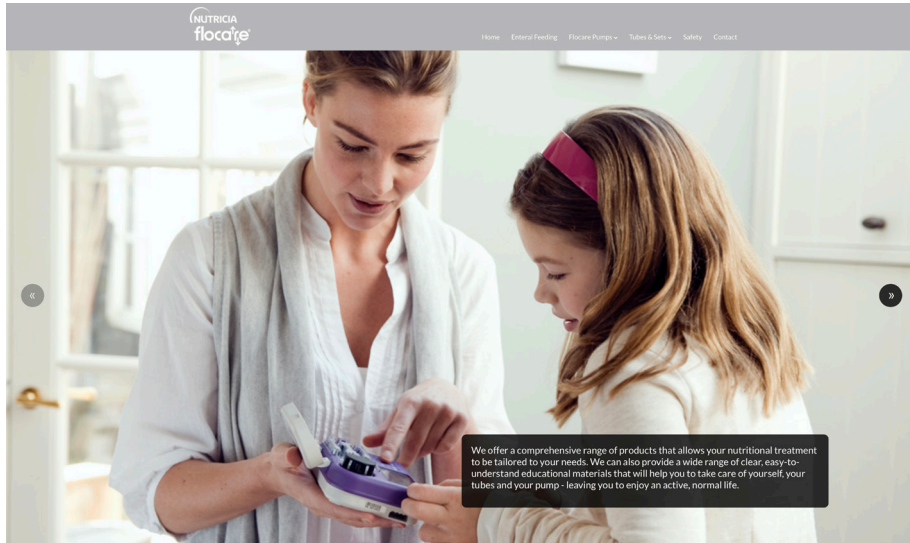

# FLOCARE INFINITY GIVING SETS

The Flocare Infinity giving set range have the following features to promote safe and easy use

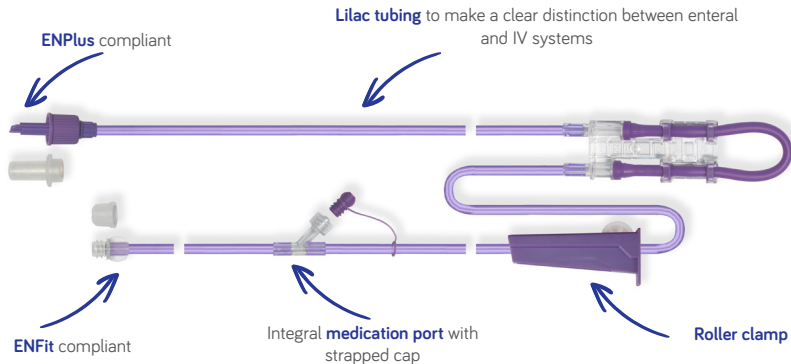

Patented anti-free flow device for additional patient safety





Single patient use

Medical grade PVC

Latex and kink free


Individually packed



Flocare Infinity Pack giving set	Flocare Infinity mobile giving set	Flocare Infinity gravity pack giving set	Flocare Infinity top-fill reservoir with integral giving set
			
<p>For use with all Nutricia Pack and OpTri presentations</p>	<p>For use with all Nutricia Pack and OpTri presentations. Shorter (1580mm) than standard giving sets (2300mm)</p>	<p>For gravity feeding For use with all Nutricia Pack and OpTri presentations</p>	<p>For the administration of decanted enteral feeds &amp; water 1.3 litres volume capacity</p>
<p><b>ORDER CODES</b> (30 units per case): 95349 (no med port) 95347 (with med port)</p>	<p><b>ORDER CODES</b> (30 units per case): 86484 (no med port) 101586 (with med port)</p>	<p><b>ORDER CODE</b> (30 units per case): 86460</p>	<p><b>ORDER CODE</b> (30 units per case): 90011 (no med port)</p>

# NASOGASTRIC ENTERAL FEEDING TUBES

Polyurethane (PUR) nasogastric enteral feeding tubes

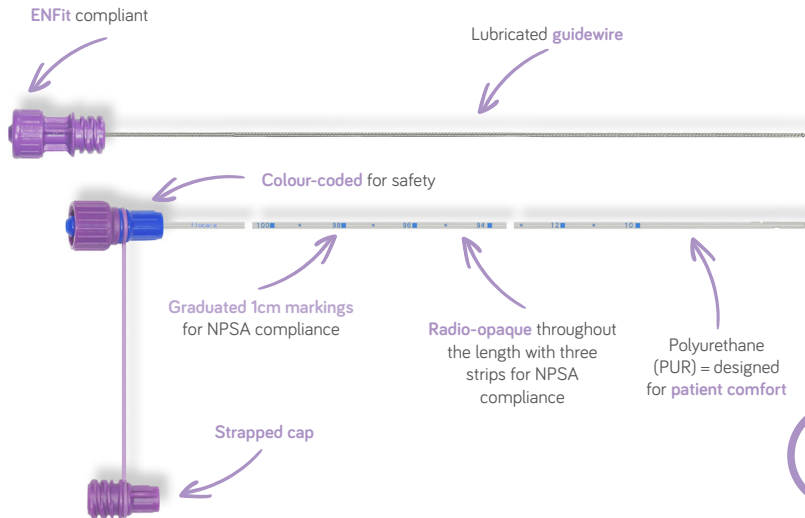

Rounded tip for safe insertion

Feeding ports designed to minimize blockages

Main material: medical grade polyurethane (except guidewire)





Sterile, latex free

CH **6** **8** **10**

Individually packed




## NASOGASTRIC ENTERAL FEEDING TUBES

CH6 60CM	CH6 110CM	CH8 110CM	CH10 110CM
			
10 units per case	10 units per case	10 units per case	10 units per case
ORDER CODE: 94567	ORDER CODE: 94568	ORDER CODE: 94569	ORDER CODE: 94570

# REPLACEMENT BALLOON GASTROSTOMY TUBES

The gastrostomy tube for replacement of PEG or an existing gastrostomy tube

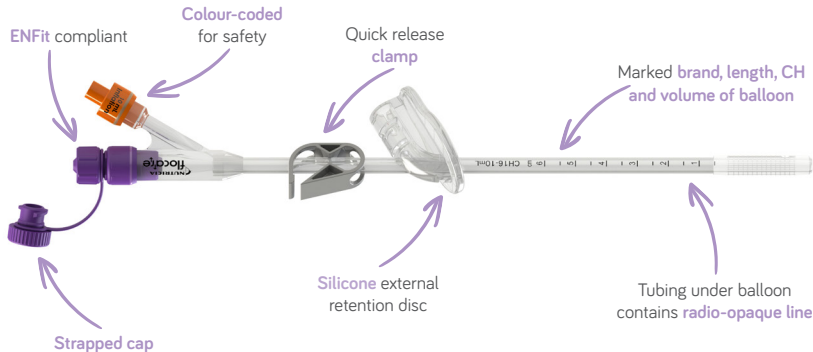

Easy to use, easy to fit

Kink-free tubing

Designed for maximum patient comfort

Sterile, latex free







Main material: medical grade silicone

CH 10 12 14 16 18 20

Individually packed




# REPLACEMENT BALLOON GASTROSTOMY TUBES

CH10	CH12	CH14	CH16	CH18	CH20
					
10	12	14	16	18	20
Volume of balloon = 3ml	Volume of balloon = 5ml	Volume of balloon = 5ml	Volume of balloon = 10ml	Volume of balloon = 15ml	Volume of balloon = 15ml
2 units per case	2 units per case	2 units per case	2 units per case	2 units per case	2 units per case
ORDER CODE: 178129	ORDER CODE: 178130	ORDER CODE: 178131	ORDER CODE: 178132	ORDER CODE: 178133	ORDER CODE: 178134



# GASTROSTOMY BUTTON

Low profile gastrostomy button for use in a well-established gastrostomy tract for feeding and/or medication

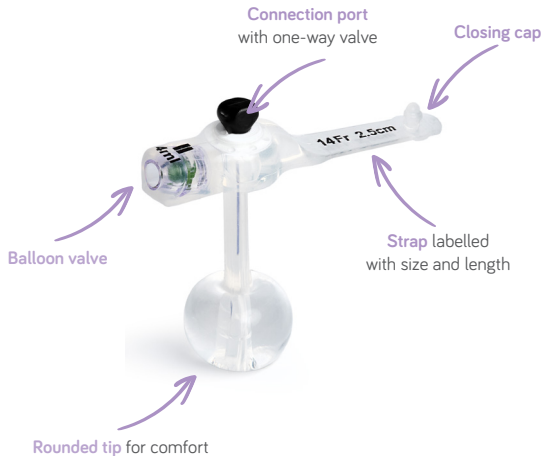

Specific size introducer for easy and secure placement

Internal silicone retention balloon

Main material: silicone

Sterile, latex free

CH 

12	14	16	18	20	24
----	----	----	----	----	----


Individually packed



GASTROSTOMY BUTTON & STOMA MEASURING DEVICE								
Diameter	Length	Order Code	Diameter	Length	Order Code	Diameter	Length	Order Code
CH12	0.8cm	152911	CH14	3.0cm	152987	CH18	2.0cm	153087
CH12	1.0cm	152913	CH14	3.5cm	152986	CH18	2.3cm	153086
CH12	1.2cm	152983	CH14	4.0cm	152985	CH18	2.5cm	153085
CH12	1.5cm	152982	CH14	4.5cm	152984	CH18	2.7cm	153084
CH12	1.7cm	152981	CH14	5.0cm	153048	CH18	3.0cm	153083
CH12	2.0cm	152980	CH16	1.0cm	153046	CH18	1.0cm	153033
CH12	2.3cm	152979	CH16	1.2cm	153045	CH18	3.5cm	153082
CH12	2.5cm	152978	CH16	1.5cm	153044	CH18	4.0cm	153081
CH12	3.0cm	153000	CH16	1.7cm	153043	CH18	4.5cm	153080
CH12	3.5cm	152999	CH16	2.0cm	153042	CH20	1.5cm	153077
CH12	4.0cm	152998	CH16	2.3cm	153041	CH20	1.7cm	153076
CH14	0.8cm	152996	CH16	2.5cm	153040	CH20	2.0cm	153075
CH14	1.0cm	152995	CH16	2.7cm	153039	CH20	2.3cm	153074
CH14	1.2cm	152994	CH16	3.5cm	153037	CH20	2.5cm	153073
CH14	1.5cm	152993	CH16	4.0cm	153036	CH20	3.0cm	153100
CH14	1.7cm	152992	CH16	4.5cm	153035	CH20	4.5cm	153098
CH14	2.0cm	152991	CH18	1.0cm	153033	CH24	3.5cm	153090
CH14	2.3cm	152990	CH18	1.2cm	153032	Gastrostomy Button Stoma Measuring Device		
CH14	2.5cm	152989	CH18	1.5cm	153089			
CH14	2.7cm	152988	CH18	1.7cm	153088			

# BENCHMARK NASO-INTESTINAL TUBES

For duodenal and jejunal feeding

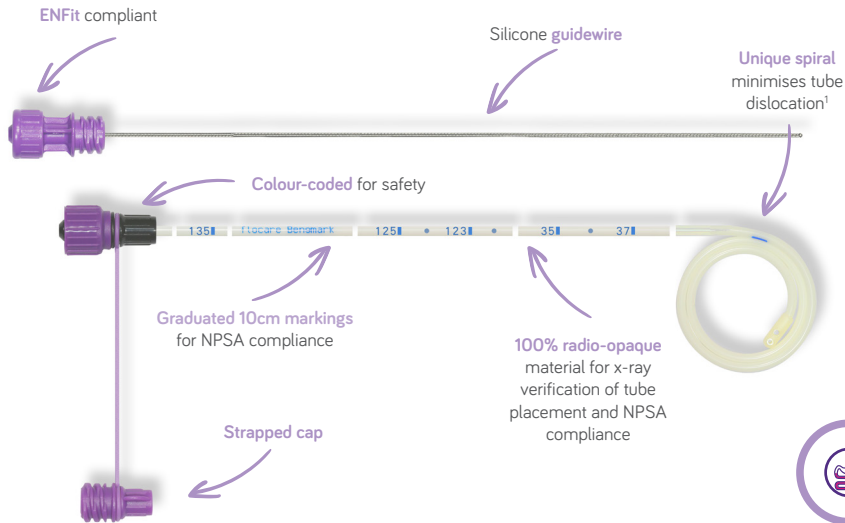

Spontaneous transpyloric passage<sup>1</sup>

Reduced need for endoscopic and fluoroscopic placement<sup>2</sup>

Main material: medical grade polyurethane (except guidewire)

Easy aspiration for tube verification

Unweighted tube

Hydromer<sup>®</sup> coated tip

4 lateral outlet ports designed to minimise blockages

Sterile, latex free

CH **8** **10**

Individually packed



# BENCHMARK NASO-INTESTINAL TUBES

CH8 145CM



8

3 units per case

**ORDER CODE:** 94824

CH10 145CM




10

3 units per case

**ORDER CODE:** 94825

# FLOCCARE INFINITY GO FRAME

Designed to hold the Flocare Infinity pump and any presentation of Nutricia feed or Flocare container

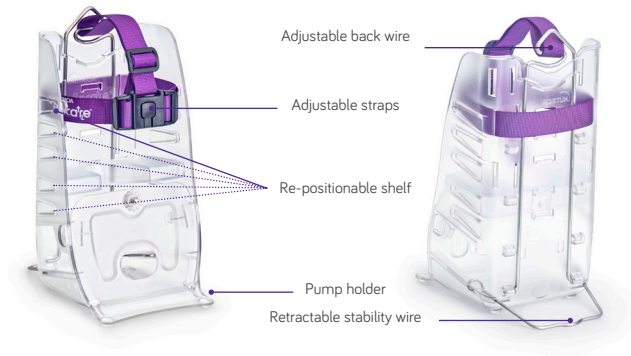

For bedside and mobile use

Compatible with Flocare giving sets and Flocare containers only

Will also fit in patients own bag

Supplied as single units

**ORDER CODE:** 188229



# FLOCARE INFINITY GO BAG

Compatible with the Flocare Infinity Go Frame to enable mobile feeding

Designed to hold the Flocare Infinity pump and any presentation of the Nutricia feed pack or Flocare container



Enough space for a bottle giving set

Elasticated Velcro® strap to secure any bottle size

Available in black

For adults and paediatrics

Supplied as single units

**ORDER CODE:** 64432



# FLO CARE INFINITY Z-STAND

Designed to fit the Flo care Infinity pump and any presentation of the Nutricia feed pack or Flo care container



Suitable for patients at home

Supplied as single units

**ORDER CODE:** 40361



## FLOCCARE INFINITY BAG\*

Compatible with the Floccare Infinity Go Frame to enable mobile feeding

Designed to hold the Floccare Infinity pump and any presentation of the Nutricia feed pack or Floccare container



\*This is a chargeable item

Available in adult (black) and  
paediatric (red/blue) versions

Compact and discrete

Supplied as single units

### ORDER CODES:

Adult (black) 44497

Paediatric (red/blue) 44496





## FLOCARE CONTAINER

For decanting feeds and administration of water.

Not suitable for microwave use. Maximum temperature 65°.



500ml container

1 litre container

Maximum capacity 520ml

Maximum capacity 1.2 litres

There are now reusable options available for decanting. Please visit [nutricia.co.uk/hcp/resource-centre/reusable-bottles-patient-user-guide.html](https://nutricia.co.uk/hcp/resource-centre/reusable-bottles-patient-user-guide.html) for further information

Semi collapsible

Single patient use only

Sterile, latex free, low density polyethylene (LDPE)

10 units per case

### ORDER CODES:

500ml: 40441

1 litre: 40455



# FLOCARE UNIVERSAL ADAPTOR

Use alongside the Flocare Infinity Pack or Flocare Infinity Mobile giving sets



Fits any 40mm diameter bottle

30 units per case

**ORDER CODE:** 189114



# BOLUS ADAPTOR

Connector for bolus feeding or decanting from Nutricia Packs and OpTri bottles, making it easy to pour directly from a feed into a syringe



Packs and OpTri bottles can be safely resealed between feeds

Sterile, latex free


30 units per case

**ORDER CODE:** 190390



## BOLUS SET

Allows decanting of a tube feed into barrel of syringe to bolus feed

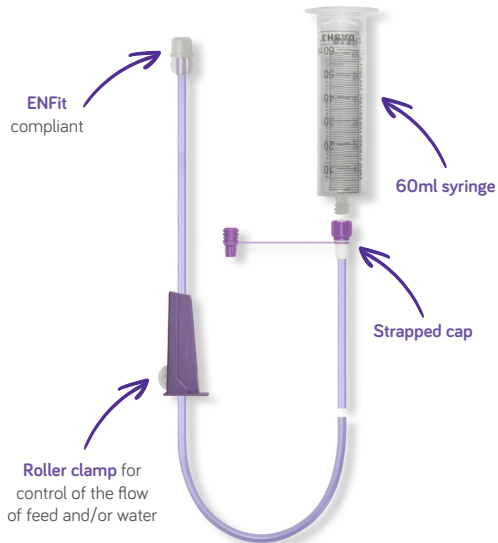

Single patient use only

Latex free

30 units per case

**ORDER CODE:** 89752



# BOLUS SET HANGER

Connects the syringe to the z-stand when bolus feeding




Supplied as single units

**ORDER CODE:** 145585



## EXTENSION SET

Extends the length of any giving set or feeding tube by 100cm

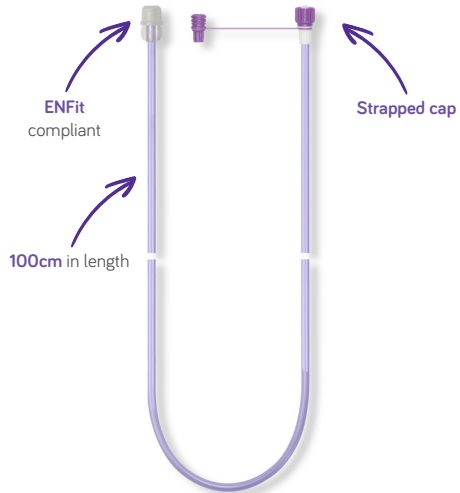

Single patient use only


40 units per case

ORDER CODE: 91393



## BUTTON EXTENSION SET

Right angle lock for secure connection to button. Compatible with Mic-Key® and Mini-One Low Profile Gastrostomy Tube

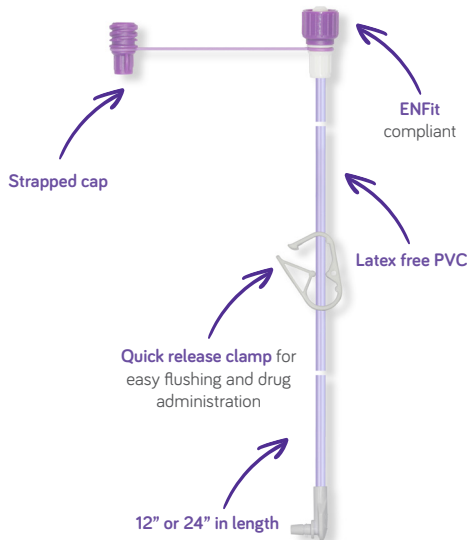

Up to 2 weeks usage, [refer to cleaning instructions at back of this document](#)

Available in 12" and 24"

5 units per case

### ORDER CODES:

91398 (12")

182967 (24")



## 2-PACK CONNECTOR

Connects 2 x Nutricia Packs or OpTri bottles of feed to one Flocare giving set and pump



**Quick release clamps** to close flow from either Pack/OpTri bottle simultaneously or independently



Sterile, latex free

Main material: medical grade PVC

30 units per case

**ORDER CODE:** 69915










## EXAMPLE OF EQUIPMENT NEEDED FOR CONTINUOUS FEEDING USING THE FLOCARE INFINITY PACK GIVING SET

Flocare Infinity pump	Flocare Infinity Pack giving set	Nutrison Pack or OpTri bottle enteral feed	Nasogastric enteral feeding tube	Flocare Infinity Go 4pr11156*G Frame
				

**For mobile feeding you will also need:** Flocare Infinity Bag/Go-Bag and Flocare Infinity Go-Frame

Example only — clinical judgement should be used when deciding on equipment required for your patient

## EXAMPLE OF EQUIPMENT NEEDED FOR BOLUS FEEDING

Flocare bolus set*	Flocare bolus adaptor*	Flocare bolus set hanger and Z-Stand	Enteral feed	Nasogastric enteral feeding tube
				

\*For further information on how to use the Flocare bolus set and Flocare bolus adaptor go to [nutriciahcp.com](https://www.nutriciahcp.com)

Example only — clinical judgement should be used when deciding on equipment required for your patient

## EXAMPLE OF EQUIPMENT NEEDED FOR USING THE 2-PACK CONNECTOR

Flocare Infinity pump	Flocare Infinity Pack giving set	2 x Nutrison Packs or OpTri enteral feeds	Flocare 2-Pack connector*	Nasogastric enteral feeding tube	Flocare Infinity Z-Stand
					

\*For further information on how to use the Flocare 2-Pack connector go to [nutriciahcp.com](http://nutriciahcp.com)


Example only — clinical judgement should be used when deciding on equipment required for your patient

# CLEANING INSTRUCTIONS FOR THE BUTTON EXTENSION SET

To reduce the risk of contamination wash the button extension set thoroughly after every use, using the following instructions.

- Close the clamp and turn the extension set anti-clockwise to match up the markings. Gently pull the extension set to remove it from the button
- Open the clamp and wash thoroughly in warm soapy water
- Using a brush, clean the connectors to remove any build up in the threads
- Flush with at least 60ml sterile/cool boiled water to remove any soap remnants
- **If using for more than 7 days (maximum 14 days)** flush with at least 60ml sterilising solution, suitable for use with plastic products. Rinse thoroughly with sterile/cool boiled water to remove any sterilising solution
- Allow to air dry in a hanging position. Store in an air tight container or zip lock bag until next use.

**Before each use check the connectors and tube for cracking, dislocation, or discolouring. If any of these are observed a new extension set must be used.**

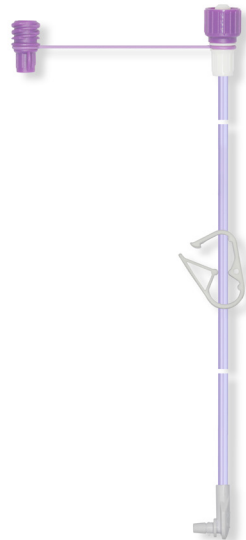

# Nutricia Resource Centre

## Healthcare Professional Helpline

Dietetic-led nutritional advice for healthcare professionals

01225 751098

## Patient and Carer Helpline

Service dedicated to the needs of patients and carers

03457 623653

Email: [resourcecentre@nutricia.com](mailto:resourcecentre@nutricia.com)

Or find us at [www.nutriciahcp.com](http://www.nutriciahcp.com)

Nutricia Limited  
Newmarket Avenue, White Horse Business Park  
Trowbridge, Wiltshire BA14 0XQ

[nutricia.co.uk](http://nutricia.co.uk)  
[tubefeeding.co.uk](http://tubefeeding.co.uk)  
[nutriciaflocare.com](http://nutriciaflocare.com)

AMN0753-03/23

The information contained within this compendium is correct to the best of our knowledge at the date of going to press.

For up to date information on all our products, or to provide comments or feedback, please visit our website or contact either the Nutricia Resource Centre or your local sales representative.

